

SPEX Seating Decision Table

Person

Symmetrical/ <u>Mild</u> Postural Needs			Asymmetrical <u>Moderate</u> Postural Needs			Asymmetrical <u>Complex</u> Postural Needs		
<i>Cushion</i>	<i>Back support</i>	<i>Accessories</i>	<i>Cushion</i>	<i>Back support</i>	<i>Accessories</i>	<i>Cushion</i>	<i>Back support</i>	<i>Accessories</i>
Vigour standard contour	Vigour Back Support	Consider micro-cube foam upgrade for maximum breathability for control of temperature and moisture Laterals can be attached to Vigour Mid and Hi Head support Arm supports	Spex standard contour	Mantaray	Consider Tessellated Positioning Kit upgrade for positioning Consider micro-cube foam upgrade for breathability, temperature and moisture control Lateral Trunk Supports Several lateral Support Pads	Spex high contour	Spex SuperShape	Axial Lateral Trunk Supports Several lateral Support Pads Head support likely needing zoom mount Axial Medial Thigh Support Axial Lateral thigh support Arm support Anterior Pelvic Positioning Anterior Trunk Positioning Several hardware options (fixed, swing-away, quick release, swing-down, ...) Several positioning wedges available
	Lo single mount			Laterals included Some adjustment in positioning through laterals Dual mount or quad mount – adjustable or strength Positioning wedges included				
Vigour high contour	Hi		Strategic contouring pads included in kit	Comfi Back	Head support Medial Thigh Support Lateral Thigh support Arm supports Anterior Pelvic Positioning Anterior Trunk Positioning	Strategic contouring pads included in kit Removable lateral rigidizers	Dual mount or quad mount – adjustable or strength	
	Mantaray		Spex super high contour (for extra immersion)	Spex back	Several hardware options (fixed, swing-away, quick release, swing-down, ...)			
	Laterals included Some adjustment in positioning through laterals Dual mount or quad mount – adjustable or strength Positioning wedges included		“T” or rectangular shape Cube contouring cell system allows for individualized configurations. Cubes to be removed/added as needed (incl. in kit). Laterals can be attached Dual mount or quad mount – adjustable or strength Positioning wedges included	“T” or rectangular shape Cube contouring cell system allows for individualized configurations. Cubes to be removed/added as needed (incl. in kit). Laterals can be attached Dual mount or quad mount – adjustable or strength Positioning wedges included				