



Active Mobility

One Way Slide

One Way Slides are designed to easily assist in the correct positioning of a person in the back of a chair and to prevent them slipping forward.

The sliding surfaces use a combination of low friction and strips of easy sliding one way materials. For the outer fabric select a non slip cover for use on slippery surfaces (essential) or for regular fabric surfaces: or select a velvet cover with a light fill for added comfort, but only for use on fabric surfaces where its fibres can grip.

One Way Slide models include:



Lite Velvet (2764012) – a single layer One Way Slide for use to help position a person on pressure cushions if they do not have severe decubitus problems (shown left) or for positioning on a swivel cushion as our Twist Assist® to help on entering or leaving vehicles (shown right).



Long single layer models that can be folded with their handle placed through the back of an open back chair allowing the carer to use their strong leg muscles to easily position the person at the back of the chair.

2764016 Velvet (shown left) or 2764015 Nonslip (shown right)



Continuous loops that can be used on open or closed back chairs. Available in regular or bariatric sizes, for wheelchairs (shown left), for lounge or recliner tilt chairs (shown right) or for tub or fall out chairs (shown far right). For Velvet and NonSlip sizes refer below.



**Nonslip Cover – use any surface, but essential for use on slippery surfaces.
Velvet Cover – use only on a fabric surface where it can grip.**

Single layer Lite for Pressure or Swivel cushions 2764012 –37 x 43 cm

Long open single layer, 130 x 40cm foot operated - open back chair Nonslip 2764015, Velvet 2764016

Continuous Loops

Small 37 x 43 cm Nonslip 2764013; Velvet 2764014
Regular 44 x 45 cm Nonslip 2764010
Velvet 2764011
Large 45 x 50 cm Nonslip 2764017
Bariatric Nonslip:- 50 x 60 cm 2764018;
55 x 70 cm O5570NT; 55 x 80 cm 2764020 For
Fall out Chairs Nonslip 80 x 60 cm 2764021 For
Beds 100 x 80 cm 2764022 (not shown)



Wash at a maximum of 60°C
Do not use fabric softener or chlorine.

Distributor