



Active Mobility

Savaria Portable Ceiling Lift **PL**

OWNER'S MANUAL



To Be Retained by Owner After Installation by Authorized Dealer

Rev. 003 03-m12-2019

STANDARD NOTATIONS

The following notations may be used throughout this manual to emphasize important safety information, mechanical concerns, and other important information. Please review and follow all of these messages.



DANGER

Danger messages indicate an imminently hazardous situation, which, if not avoided, results in death or serious injury. All danger messages feature a standard ISO safety alert symbol followed by the signal word **DANGER** in capitalized black lettering on a red background.



WARNING

Warning messages indicate a potentially hazardous situation, which, if not avoided, could result in death or serious injury. All warning messages feature a standard ISO safety alert symbol followed by the signal word **WARNING** in capitalized black lettering on a dark yellow background.



CAUTION

Caution messages indicate a potentially hazardous situation, which, if not avoided, could result in minor injuries. All caution messages feature a standard ISO safety alert symbol followed by the signal word **CAUTION** in capitalized black lettering on a yellow background.

NOTE

Note messages provide information, such as reminders, general information about a previous statement, or additional guidelines that do not fit into the flow of the preceding text. All note messages include the signal word **NOTE** in capitalized white lettering on a blue background.

CONTENTS

1. OVERVIEW	5
Intended Use	5
Product Life	5
Using This Manual	5
2. SAFETY INSTRUCTIONS	6
To Ensure Safety	6
Battery and Charger Safety	7
Fire and Explosion Prevention	7
Shock Prevention	8
3. DESCRIPTION	9
Exterior Views	9
Lift Controls and Connectors	10
Hand Control	10
Charger	10
Slings	11
Symbols	13
4. SPECIFICATIONS	16
Dimensions	17
5. USING THE LIFT	18
Before Transfer	18
Transferring the Person	18
Emergency Lowering	28
Emergency Brake	29
Charging the Battery	30
6. MAINTENANCE	31
Operator Maintenance Schedule	32
Dealer Maintenance Schedule	33
Inspection and Cleaning	34
Cleaning the Sling	35
7. TROUBLESHOOTING	36
8. ELECTROMAGNETIC COMPATIBILITY	37
Electromagnetic Compliance	37
Electromagnetic Immunity	38

1. OVERVIEW

Intended Use

The Monarch Portable Ceiling Lift is designed for use by caregivers and health care professionals to lift and transfer a person in a homecare, assisted-living or hospital environment. Transfer must be performed by a qualified operator as described in this manual. **Do NOT use the lift for any other purpose.**

Product Life

The lift is designed for a useful life of 10 years after which time it must be replaced. To ensure this life span, you must perform the preventive maintenance as specified in the *Maintenance* section in this manual.



WARNING

Savaria Concord Lifts, Inc. cannot ensure complete safety for a portable ceiling lift that has exceeded its useful life. Wear may cause failure of a part and lead to a patient fall.

The expected life for other parts such as slings, batteries, fuses, straps, and cords is dependent upon the proper care and use of the individual items. The items must be maintained as described in their accompanying documentation and in the *Maintenance* section in this manual.

Using This Manual

To ensure safe operation of the lift, read this entire manual carefully, especially the *Safety Instructions*, before installing, using or maintaining the lift. Failure to comply with all WARNINGS in this manual may result in injury. If there is anything you don't understand, contact your Authorized Savaria Dealer for further details.

Keep this manual with the lift and ensure all operators are fully trained in the use of the lift as described in this manual.

2. SAFETY INSTRUCTIONS

Keep these *Safety Instructions* with the lift at all times. Read this entire manual before installing, using or maintaining the lift.



WARNING

Do NOT use the lift for any purpose other than to transfer a person. To ensure safety of the person being transferred and the operator, follow all Safety Instructions.

Take the necessary precautions to avoid any risk of entanglement posed by the lifting strap or hand control cable.

To Ensure Safety

- The lift has a safe working load of 200 kg (440 lb) or 130 kg (286 lb). Do NOT exceed this load. Refer to the nameplate tag for the Maximum Load allowed.
- The lift shall be installed in a track system with equal or greater capacity. Never install a lift into a track not having sufficient loading capacity.
- The lift must be installed by an Authorized Savaria Dealer.
- Ensure the lift is used only with a Savaria ceiling track system. If the track system is not from Savaria, ensure the track system is properly assessed by a competent authority for fit, function, and safety.
- The track system must be installed by a trained and Authorized Savaria Dealer.
- Operators must be fully trained before using the lift.
- Ensure the sling is intended for use with this lift and can handle the weight of the person. Ensure the sling is not damaged in any way and the sling straps are in good condition and attached properly.
- Check with a qualified health professional to ensure the person to be transferred has been deemed suitable for transfer.
- Take extra care with a person who is connected to electrodes, catheters or other medical devices to ensure safe transfer.
- Take care to avoid impact during transfer.
- Keep all components of the lift clean and dry.
- Follow the lifting procedures provided in this manual.

