

# ATMOS

ULTRA LITE



*Azure*

USER MANUAL

TRANSIT | SELF-PROPEL

# USER MANUAL CONTENTS

<b>Preface &amp; Introduction</b>	<b>3</b>
<b>Get to know your Atmos</b>	<b>4</b>
<b>Safety &amp; Inspection</b>	<b>6</b>
Notice to Caregivers	6
Safety Notice	6
Pre-Ride Safety Inspection	7
<b>Setting up your wheelchair</b>	<b>8</b>
<b>Safe Working Load</b>	<b>10</b>
Maximum Weight Limits	10
<b>Wheelchair Operation</b>	<b>11</b>
Operating Tips	11
How to Brake	11
Manual Wheel Locks & Hand Brake	12
<b>Wheelchair Maintenance</b>	<b>13</b>
<b>Warranty</b>	<b>15</b>

# ATMOS

Congratulations and thank you for your purchase of an ATMOS Ultra-lite Manual Wheelchair by Azure. We are sure you will find your new wheelchair a pleasure to use.

This manual is applicable to Atmos Transit and Self-Propel wheelchair models. Before using your new Atmos wheelchair, we recommend that you read this user manual which provides helpful information on the safe usage of this product as well as care instructions and your warranty.

To avoid any risk of harm or injury, it is important that you check the safe working load of your wheelchair and to read the safety instructions carefully to avoid misuse and hazards. Please be aware that Incorrect use of your Atmos wheelchair will void warranty.

For future reference, please store this booklet in a safe place.

# GET TO KNOW YOUR ATMOS

- 1 Hand grip
- 2 Brake lever (Transit only)
- 3 Push handle locks
- 4 Phone Pocket
- 5 Removable/ washable cushion
- 6 Removable leg rests
- 7 Foot Rests
- 8 Front wheels
- 9 Transit Back Wheel (Transit only)
- 10 Wheel Release Button (Self-Propel only)





# SAFETY AND INSPECTION

## NOTICE TO CAREGIVERS

Please be sure to pay careful attention to all warnings and follow all instructions set out in this manual. It is important that you pay close attention to all warnings in this section. If you fail to do so, you may fall, tip over or lose control resulting in severe injury to you and others

## SAFETY NOTICE

1. Ensure adequate time to learn how your Atmos wheelchair operates before you begin using.
2. Always be aware of your surroundings when operating this product.
3. The Wheelchair is designed for a single User/Occupant only. More than one user may result in injury to you and others.
4. Do not modify your wheelchair in any way unless authorized by Azure
5. Ensure any repairs or adjustments are carried by an Authorized agent of this product.

## PRE-RIDE SAFETY INSPECTION

We recommend that you perform a safety check before each use to make sure your wheelchair operates smoothly and safely. Before operating your wheelchair you should be trained in the safe use of this wheelchair by your authorized provider.

Perform the following inspections prior to using your wheelchair:

6. Check all wheels are in full operating condition, rotating freely & absent from damage, excessive tyre & castor wear and making full contact with the ground.
7. Check all hand brakes & park operation.
8. Check swing away leg rests are locked into position.



# SETTING UP YOUR WHEELCHAIR





## WHEELCHAIR SET-UP

Remove your wheelchair from the packaging materials first and confirm all components are present. To set up your wheelchair start by releasing your wheelchair from the fold lock position by unbuttoning the material tab. Then, lift the rear push handles on the back canes to move the seat back into position and ensure it locks into place with the two back hinges.

Next press down on the outside seat rails or armrest and your product will fully unfold.

Finish the setup of your new wheelchair by installing the Velcro attached seat back and seat bottom overlay pieces, and installing the front legrests. Please note that the overlay pieces fit only on the seat bottom or the seat back and the Velcro pieces connect only one way. The front legrests are side-specific items, either Left or Right.



# PRE RIDE SAFETY CHECK

## SAFE WORK LOAD

1. The Atmos Ultra-lite Transit and Self-Propel wheelchair range has a Safe Work Load (SWL) of 136kg
2. Exceeding the product weight limit may damage the seat, frame, fasteners, upholstery and may result in injury to you or others.
3. Exceeding the Safe working Load of 136kg will void the warranty of this product.
4. Stay within the specified weight capacity of your wheelchair. The manufacturer will not be held responsible for injuries and/or property damage resulting from failure to observe weight limitations.
5. Do not carry passengers in your wheelchair. Your wheelchair is designed for the carriage for one person only. Exceeding this capacity is dangerous and may affect the center of gravity.

Model	Maximum User Weight
<b>ATMOS</b> TRANSIT	136kg
<b>ATMOS</b> SELF-PEPEL	136kg

# WHEELCHAIR OPERATION

## OPERATING TIPS

1. Do not lean forward when operating, especially on stairs and uneven surfaces.
2. When turning, reduce your speed and maintain a stable center of gravity.
3. When on any incline or decline, never allow the wheelchair to roll freely while seated in it or standing next to it.

## HOW TO BRAKE

**Atmos Self-Propel** - To stop the wheelchair, apply resistance to the rear wheel hand rims until the wheelchair comes to a complete stop.

**Atmos Transit** - To stop the wheelchair, the rear caregiver hand brakes must be squeezed until the wheelchair comes to a complete stop.

## MANUAL WHEEL LOCKS AND HAND BRAKE

Your Atmos Self-Propel wheelchair is equipped with two manual rear wheel brakes which can lock into place, one mounted on each rear wheel. Your Atmos Transit wheelchair and the rear caregiver handbrakes is mounted onto the rear handles. Both sets of brakes can lock into place.

The Atmos Self-Propel manual wheel locks help prevent your wheelchair from moving when parked. After you bring your wheelchair to a complete stop, engage both manual rear wheel locks. If you have any questions about the operation of your manual wheel locks, contact your authorized provider.

The Atmos Transit wheelchair locks in to place by pushing down on the rear caregiver handbrakes.

It is not recommend that you remain parked on an incline for a long period of time as your wheelchair may slide.



# WHEELCHAIR MAINTENANCE

It is important to perform proper care of your Atmos wheelchair in order to maintain the life and quality of your product and to ensure your product is functioning properly.

1. Check for loose or missing screws, nuts, and bolts. Replace or tighten them as necessary.
2. Check for flat spots on solid tires. Flat spots could adversely affect stability.
3. Check all manual wheel locks and brakes. Make sure they are working properly.
4. Check the caster forks for damage or fluttering which indicates that they may need to be adjusted or have the bearings replaced.
5. Keep your wheelchair clean and free of foreign material, such as mud, dirt, hair, food, drink.
6. You may wash your seat by hand wash only with warm water and a PH neutral washing powder.



# WARRANTY

The warranties herein specified shall not apply if the failure of any component of this product is caused by the purchaser or its agent due to accident, neglect, misuse, improper transportation or operation contrary to the manufacturer's operating instructions, or if any person other than the manufacturer's authorised personnel shall modify, adjust, or repair such product or perform any related service on it. It must be understood that the purchaser assumes full responsibility for the overall safe operation in the environment for which the product is intended to function.

**Frame:** The manufacturer covers the frame of its mobility chair under normal use for a 2 year warranty period. The manufacturer will, at its sole discretion, repair or replace any warrantable frame that fails within the 2 year warranty period.

**Upholstery:** Upholstery and arm pads under normal operating conditions carry a 12 month warranty period. The manufacturer, at its sole discretion, will replace any warrantable components that fail within the 12 month warranty period.

**Castors & Rear Wheels:** Front castors and rear wheels with associated sub-assemblies under normal carry operating conditions carry a 12 month warranty period. The manufacturer, at its sole discretion, will replace any warrantable components that fail within the 12 month warranty period.

---

**To ensure the correct warranty support, please write down the serial number that is attached on the under side of the frame.**

Model	Serial Number
<b>ATMOS</b> TRANSIT	
<b>ATMOS</b> SELF-PROPEL	

# ATMOS

Unit 8, 110-120 Silverwater Rd  
Silverwater NSW 2128  
Ph: (02) 9649 2111

For more information, visit:  
[activemobility.com.au](http://activemobility.com.au)