

rolio

Azure



seat walker
user manual

rolio | **rolio+**

user manual

contents

Preface & Introduction	3
Parts Description	4
Important Safety Information	6
- Maximum Weight Limits	
Usage Instructions	8
- How to unfold rollo	
- How to fold rollo	
- How to brake	
- How to attach/remove bag	
- How to adjust the handle height	
- How to use rollo as a seat	
- How to adjust the backrest	
Maintenance & Care	12
- Frame maintenance	
- Bag & Seat maintenance	



Congratulations and thank you for your purchase of a rolío seat walker by Azure. We are sure you will find your new walker a pleasure to use.

This manual is applicable to rolío and rolío+ seat walker models. Before using your new rolío walker, we recommend that you read this user manual which provides helpful information on the safe usage of this product as well as care instructions and your warranty.

To avoid any risk of harm or injury, it is important that you check the safe working load of your walker and to read the safety instructions carefully to avoid misuse and hazards. Please be aware that Incorrect use of your rolío will void warranty.

For future reference, please store this booklet in a safe place.

get to know your **rolio**

- 1 Hand grip
- 2 Brake lever
- 3 Height adjustment button
- 4 Adjustable back rest
- 5 Seat
- 6 Tote bag
- 7 Front wheel
- 8 Back wheel





important
safety information
about your **rolio**



rolio+
180kg
Max User Weight



rolio
136kg
Max User Weight

Before use of your rolío, it is important that you read these instructions carefully and familiarise yourself with your new walker.

- Ensure all components of your rolío walker are correctly and securely installed
- Ensure you have sufficient strength to safely operate and control the walker and hand brakes
- Before using the walker seat, ensure the brakes are locked
- Do not unlock the brakes until you are standing
- Do not exceed the maximum weight limit of 136kg for rolío and 180kg for rolío+
- Avoid storing your rolío in direct sunlight or near heated sources, as parts exposed to heat and sunlight will heat the walker and may cause injury
- Do not place more than 4.5kg of items into the walker bag
- Do not move the walker while seated as this is a walking aid only

Model	Maximum User Weight
rolío	136kg
rolío⁺	180kg

how to use your **rolio**

how to unfold your rollo

For your convenience, your rollo comes fully assembled. To unfold your rollo, simply place it on its wheels on a flat surface, then press down on the tubes at each side of the seat until they lock into place.



how to fold your rollo

- Firmly pull the strap located on top of the seat
- Secure your rollo walker by pushing the two sides of the frame together until an audible click is heard
- Pull the handles apart to unlock



how to brake

To slow or stop your rollo while walking, squeeze both brake levers upwards to apply the brakes. Always apply brake pressure evenly. To lock the brakes while the walker is stopped, push down on each brake lever until they lock into place. Unlock the brakes by pulling up on the brake levers.



how to attach/remove your rollo bag

Attach the bag by sliding the plastic clip on the bag into the plastic connector on the frame.



how to adjust the handle height

To adjust the handle height:

1. Press and hold the adjustment button and move the handgrip up and down to the desired height
2. Release the adjustment button
3. The handgrip will automatically lock to the next possible height position
4. Adjust both sides to the same height



PRO TIP

To set up your rollo walker to the ideal height, ensure the top of the handle aligns with your wrist when you are standing upright.

how to use rollo as a seat

Before using rollo as a seat, make sure you position it on a level surface with the brakes in the locked position. Lock the brakes by pushing down on the brake levers until they lock into place. Do not lean back or to the sides when seated.



Adjusting the backrest

For your comfort, both the backrest height and depth are adjustable on your rollo walker.



To adjust the backrest height

1. Loosen the height adjustment knob below the handgrip
2. Slide the backrest up or down to the desired height
3. Tighten the knob

To adjust the backrest depth

1. Press and hold the silver adjustment button on the backrest
2. Slide the backrest in or out to the desired depth
3. Release the adjustment button and ensure it clicks securely into place



how to maintain your **rolio**

Frame maintenance

- Wipe the rolío frame with a damp cloth
- For more stubborn dirt, use a mild detergent solution with warm water
- Dry thoroughly before use to avoid corrosion
- Clean the wheels using a damp brush with plastic bristles (do not use a wire brush as this may damage the rubber or painted finish)





Bag & Seat Maintenance

- Regularly clean with a mild detergent solution and warm water, or wipe with alcohol-based disinfectants if necessary
- Dry thoroughly before use
- Avoid contact with saltwater or pools with a high level of chlorination



Unit 8, 110-120 Silverwater Rd
Silverwater NSW 2128
Ph: (02) 9649 2111

For more information, visit:
activemobility.com.au